



INVACARE® SOLARA® 3G
TILT-IN-SPACE WHEELCHAIR



Yes, you can.®

The **Invacare® Solara® 3G Tilt-in-Space Wheelchair** offers more adjustability and is easier to tilt compared to the previous generation Solara wheelchair. With a 300 lb. standard weight capacity, tilt assist adjustment and a seat pan that grows, the Solara 3G wheelchair offers everything you loved about the Solara 2G wheelchair plus much more.

Features

The unique Tilt Assist feature makes tilting easier by reducing the effort required to tilt an occupied wheelchair. With three settings- front, middle and rear- the simple Tilt Assist adjustments help to correct for uneven weight distribution.

The innovative Low Shear Recline is designed to minimize shearing forces and prevents displacement of positioning tools such as laterals and headrest.

Used in combination with tilt, recline can offer additional functional and clinical benefits to users and caregivers.

Foot-Operated Wheel Lock easily locks both rear wheels by pressing 1 of the 2 levers on the wheel lock. 20" and 24" back canes include a 10° bend option accommodating after-market back hardware while the back angle adjustment location better fits after-market seating systems. In addition, adjustable height back canes can be ordered with or without adjustable angle stroller handles.

The Seating Dynamics Dynamic Rocker Back* accommodates those with involuntary thrusts by helping to reduce pain or discomfort for the user. See www.seatingdynamics.com for complete details. For more detailed information visit www.invacare.com/solara3g

HCPCS Code: E1161†



1. Low shear recline
2. Back cane options
3. Seating Dynamics Dynamic Rocker Back
4. Foot-operated wheel lock option
5. Transport Brackets (TRBKTS)

*TRBKTS includes four factory-installed wheelchair transport brackets. TRBKTS has not been crash tested in accordance with WC19. Use these transport brackets only to secure an unoccupied wheelchair during transport. As of this date, the Department of Transportation has not approved any tie-down systems for transportation of a user while in a wheelchair, in a moving vehicle of any type. It is Invacare's position that users of wheelchairs should be transferred into appropriate seating in vehicles for transportation and use be made of the restraints available by the auto industry. Invacare cannot and does not recommend any wheelchair transportation systems.

Model #	Overall Dimensions	Seat Dimensions	Seat-to-Floor Height	Tilt Range	Adjustable Back Angle	Front Frame Angle	Tubing Diameter
SOLARA3G	Seat width + 8.5"	12"-24"W 12"-22"D	12.5"- 19" .5" increments	-5°-50°	0°, 5°, 10°, 15°, 20°, 25°, 30°	N/A	1.25"

Knee-to-Heel Range	Wheels	Casters	Total Weight**	Weight Capacity	Limited Warranty
4.5"-24"	12", 16", 18", 20", 22", 24"	4", 5", 6", 8"	73 lb.	300 lb. 400 lb. HD	5 Years Frame/ Crossbraces

** 18x18 frame with 12" wheels

Please see the Owner's Manual and Invacare Configurator for complete option/specification combinations.

† All references to HCPCS codes: Providers are responsible for determining the appropriate billing codes when submitting claims to the Medicare program, and should consult an attorney or other advisor to discuss specific situations in further detail.

Invacare Corporation
www.invacare.com

USA
One Invacare Way
Elyria, Ohio
44035-4190
(800) 333-6900

Canada
570 Matheson Blvd. E.,
Unit 8
Mississauga, Ontario
L4Z 4G4 Canada
(800) 668-5324

©2019
Invacare Corporation.
All rights reserved. Trademarks are identified by the symbols ™ and ®. All trademarks are owned by or licensed to Invacare Corporation unless otherwise noted. Dynamic Rocker Back is a brand of Seating Dynamics. Specifications are subject to change without notification. Form No. 19-090
190541

